

# HALF-SHIELD MASK



Half shield mask is a universal, reusable nose and mouth protection. It is designed to increase safety against microorganisms and other external factors, minimizing the risk of viral infections, including COVID-19.

Ideal to prevent from touching your face with dirty hands.

## Functionality:

- universal size
- comfortable and fast face application thanks to the chin support and two elastic bands that can be worn over the ears
- reusable
- lightweight and robust construction
- comfortable to use
- breathe easy
- does not interfere with the glasses and not causes their evaporation
- fast and easy assembly
- resistant to alcohol-based disinfectants
- dimensions: bottom width 142 mm, height 84 mm

Colours available:



White



Yellow




Black

\* Other colours available on request


# ASSEMBLY INSTRUCTIONS




Before the assembly of the visor, hands and components of the visor should be thoroughly washed and disinfected.

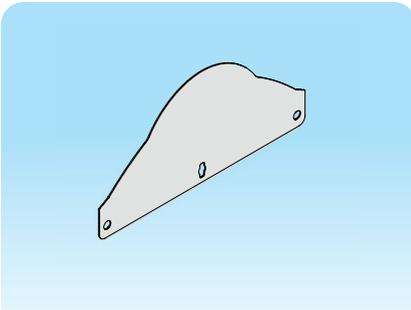


- 1 Peel the protective film off the cover (the film is on both sides of the cover)

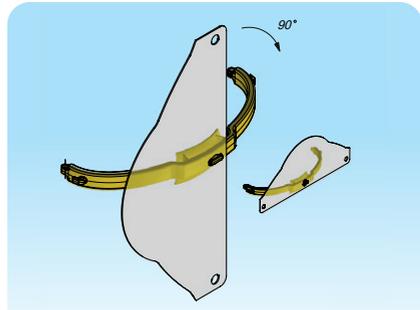

- 2 Slide the cover slot into the catch in the centre of the cover frame and rotate the cover by 90 degrees.

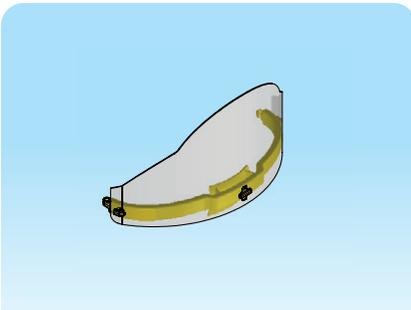

- 3 Attach the cover to the two end catches

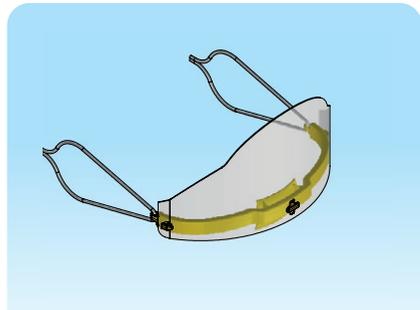

- 4 Fix the elastic straps in the holders at the ends of the frame

- There is a protective film on the half shield mask
- The film on the inside should be peeled off before assembly
- The protective film on the outside can be peeled off after assembly
- The materials of the half shield mask allow it to be disinfected with alcohol-based products
- High temperatures should be avoided due to possible deformation
- For disinfection use soft materials that do not scratch the surface of the visor

CATALOGUE NUMBER	COMPONENTS	MATERIAL
4.551.002	Shield	PET A
	Cover frame	Polypropylene
	Elastic band	Polyester + rubber

This product is not a personal protective equipment within the meaning of European Union Regulation PPER 2016/425/EU. The content of the Regulation is available at [www.gov.pl](http://www.gov.pl).